

---

# Naida™ UltraPower

## Product information

---



Naida UltraPower is a new and outstanding product line, designed for severe to profound hearing losses. It is based on the newest wireless chip technology from Phonak, CORE, offering CableFree Fitting, QuickSync coordination and the comfort of wireless connectivity. The unique combination of SoundRecover, PowerProcessing and BassBoost offers exceptional levels of audibility and clarity. With SoundRecover (non-linear frequency compression) the wearer will hear more high frequency sounds essential for speech understanding.

Feedback is effectively suppressed without gain loss, thanks to the new advanced WhistleBlock Technology. Naida also has a strong pediatric focus, with special Junior models, with mini hook, tamper-proof battery door and special software defaults. Naida is the smallest UltraPower hearing instrument ever and is WaterResistant. FM receivers are available in two versions; a WaterResistant design-integrated (ML10i), and a universal solution (MLxi). Both are based on the new and groundbreaking Dynamic FM technology from Phonak.

---

### Key features

- 6 channels
- PowerProcessing
- BassBoost
- SoundFlow Standard
- 2 programs + 2 dedicated to FM and/or T-coil
- SoundRecover
- WhistleBlock Technology
- NoiseBlock Processing
- Omnidirectional microphone
- Wind & Weather protection
- WaterResistant
- EasyFM
- EasyPhone
- DataLogging
- FM solutions: design-integrated or universal
- myPilot command center
- iCom communication interface
  
- Secure 'n Stay retention
- 675 battery
- 18 colors including 2 patterns
- CableFree Fitting
  
- AudiogramDirect
- 16 channels
- PowerProcessing
- BassBoost
- SoundFlow Advanced
- 4 manual programs (free configuration)
- SoundRecover
- WhistleBlock Technology
- NoiseBlock Processing
- Digital AudioZoom
- WindBlock Management and Wind & Weather protection
- WaterResistant
- EasyFM
- EasyPhone
- DataLogging
- FM solutions: design-integrated or universal
- myPilot command center
- iCom communication interface
- Real Ear Sound
- QuickSync
- Secure 'n Stay retention
- 675 battery
- 18 colors including 2 patterns

---

### Fitting software

- iPG version 2.0 or higher, NOAH compatible

### Hardware

- PC (IBM compatible)
- iCube CableFree Fitting or NOAHlink or HI-PRO

\*Junior models will be delivered with a mini hook and a tamperproof battery door


# Programming

---


## CableFree Fitting with iCube



1. Switch iCube on and check the Bluetooth connection to the computer (blue LED blinking slowly).

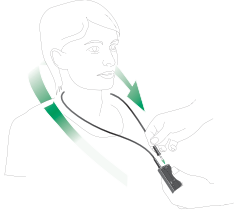

2. Put the neckloop around the customers neck and connect to the iCube.

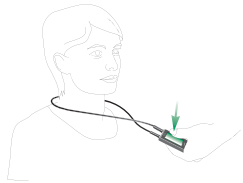

3. Check the connection between the hearing instruments and iCube by pressing the button of the iCube (middle LED lights up for 3 seconds).

---

## Via Cable (NOAHlink, HiPRO)



1. Flip open the cover for the programming socket.







2. Connect the programming plug into the socket.  
Use the programming cable CS-44 A (Phonak version) only.

## Key features

The complete range of data sheets is available on [www.phonak.com](http://www.phonak.com)


### UltraPower

	Naida III UP	Junior Naida III UP Jr	Naida V UP	Junior Naida V UP Jr
				
<b>Performance profile</b>	<b>HE7/HE7 680</b>	<b>Pediatric (mini hook) HE7/HE7 680</b>	<b>HE7/HE7 680</b>	<b>Pediatric (mini hook) HE7/HE7 680</b>
Maximum Power Output (dB SPL)				
2 cc coupler	141/135	141/135	141/135	141/135
Ear Simulator	144/139	144/139	144/139	144/139
Maximum gain (dB)				
2 cc coupler	82/75	82/75	82/75	82/75
Ear Simulator	85/80	85/80	85/80	85/80
Frequency range – 2 cc coupler (Hz)	<100 – 5000	<100 – 5000	<100 – 5000	<100 – 5000
Frequency range – Ear Simulator (Hz)	<xxx – xxxx	<xxx – xxxx	<xxx – xxxx	<xxx – xxxx
Battery size	675	675	675	675
Working current (mA)	x.x	x.x	x.x	x.x
Pediatric models with mini hook and tamperproof battery door		✓		✓

**Naida colors:** Beige, Brown, Taupe, Light gray, Black, Hightech gray, Gray, Blond, Chestnut brown, Pure transparent, Red transparent, Blue transparent, Purple transparent, Palladium/Black, Safari stripes, Savannah beauty, Light pink, Light blue

## Fitting ranges

### All models

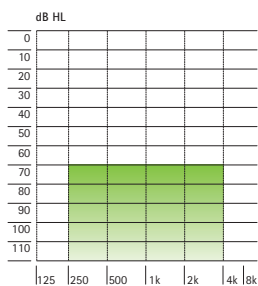


Severe to profound hearing loss,  
all audiometric configurations

## Technical description




- 1 Transparent hook
- 2 Sound outlet
- 3 Miniature microphones with Wind & Weather Protector (WaterResistant)
- 4 Program switch
- 5 Access to programming plug
- 6 Digital volume control
- 7 Serial number
- 8 Battery compartment with ON/OFF switch
- 9 Wind & Weather Protector (WaterResistant) for battery compartment
- 10 Design-integrated FM (ML10i)



## Accessories

		Naída III UP	Junior Naída III UP Jr	Naída V UP	Junior Naída V UP Jr
Digital wireless accessories	 myPilot iCom	✓	✓	✓	✓
CROSLink		✓	✓	✓	✓
Secure 'n Stay retention		✓	✓	✓	✓
Spectacle adaptor		✓	✓	✓	✓

## FM Solutions


		Naída III UP	Junior Naída III UP Jr	Naída V UP	Junior Naída V UP Jr
Design-integrated FM receiver (WaterResistant) * Tamper proof battery compartment available as an option	 ML10i*	✓	✓	✓	✓
Universal FM receivers *neckworn FM receiver	 AS10 MLxi MicroMLxS MyLink*	✓	✓	✓	✓
FM transmitters	 inspiro SmartLink SX ZoomLink EasyLink	✓	✓	✓	✓

### FM solutions colors:

**ML10i:** Beige, Brown, Taupe, Light gray, Black, Gray, Chestnut brown, Light pink, Light blue

**AS10:** Beige, Taupe, Black

## Options

		Naída III UP	Junior Naída III UP Jr	Naída V UP	Junior Naída V UP Jr
Hooks	 HE7 680 HE7 mini hook HE7 680 mini hook HE7	S 0 0 0	0 0 S 0	S 0 0 0	0 0 S 0
Volume control variations	standard locked (pediatric purpose)	S 0	S 0	S 0	S 0
EasyPhone		S	S	S	S
EasyFM		S	S	S	S
T-coil		S	S	S	S
Tamperproof battery compartment		0	S	0	S

S = Standard      0 = Optional

Note: Some options are country dependent. Please refer to your order form.